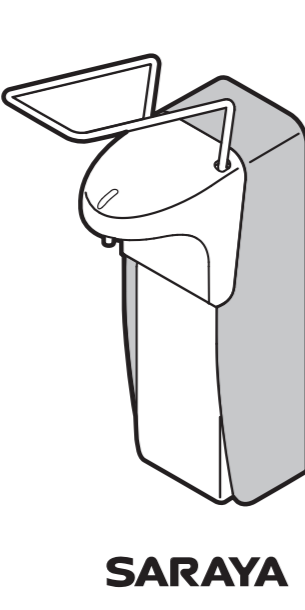
# A2(W594xH420)

## English

### Instruction Manual for SARAYA Elbow Operated Dispenser

**MDS-1000A/P, MDS-500A/P**

This unit is for use in MDS-1000A/P and MDS-500A/P Elbow Operated Dispenser for your safety, you are recommended to thoroughly read this instruction manual when using the dispenser.

**Contents**

- Safety Precautions ..... 2
- Principal Parts ..... 4
- How to use ..... 7
- Maintenance ..... 8
- Before calling for repair ..... 9
- Specifications ..... 10

**Accessories**

- Instruction manual (1)
- Screw set (3 tapping screws, 3 plugs)
- Soap pump (1)

**Safety Precautions**

**CAUTION** Improper handling will result in serious injury or death.

**WARNING** Improper handling will result in injury, PTI or death.

**CAUTION** Improper handling may result in injury, PTI or death.

**WARNING** Never install this product outdoors or in a poorly ventilated area and never use it for purposes other than those intended.

**Safety Precautions**

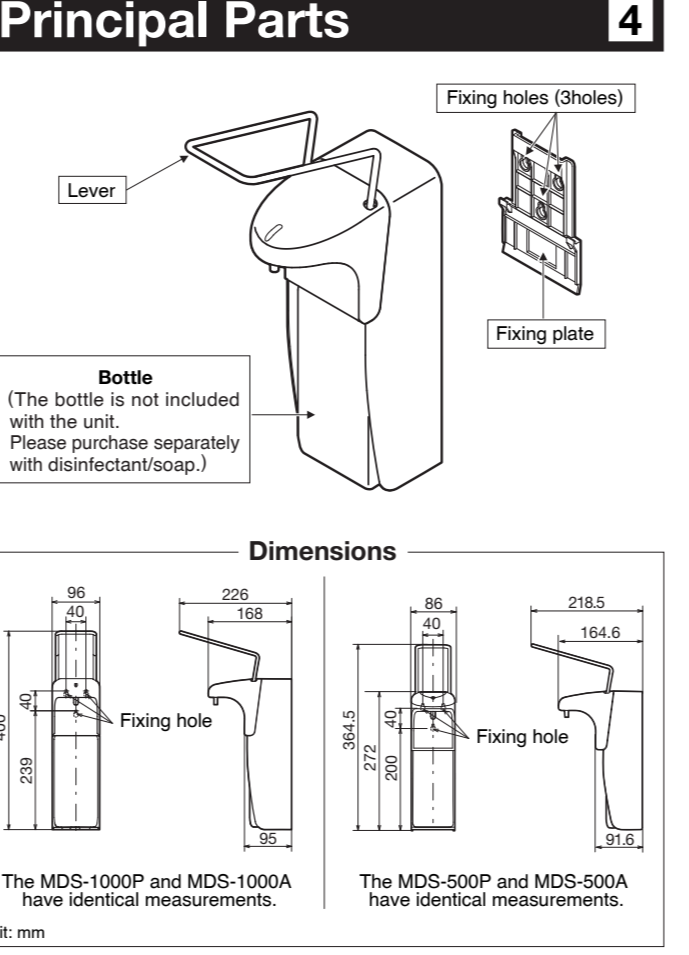
**WARNING** Stay away from fire when hands are wet with alcohol disinfectant.

**CAUTION** Do not pour or splash water on the unit.

**WARNING** Never shake the alcohol disinfectant spray.

**CAUTION** Do not touch the nozzle when the unit is in operation.

**WARNING** Do not use the unit beyond its intended specifications.

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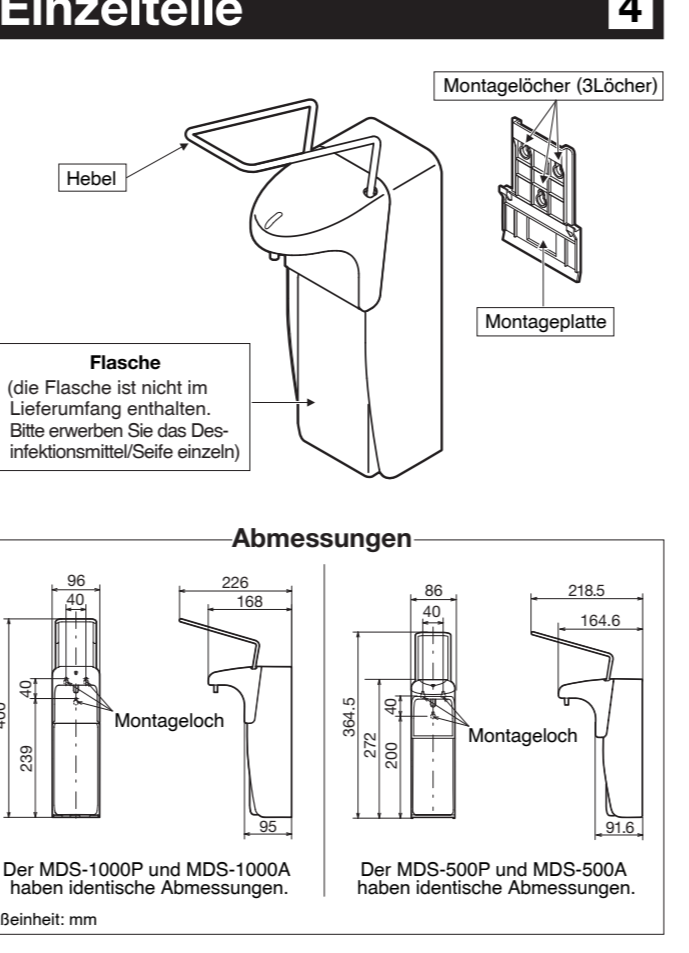
**Installation**

**Checklist**

- 1. Pump must be primed before liquid can be ejected.
- 2. Do not press the lever too strongly. Pressing too hard may cause the dispenser to malfunction.
- 3. Place the warning label on the bottle before the dispenser is in operation.

**Installation**

**Checklist**

- 1. Ensure the unit is mounted on a concrete wall.
- 2. Do not use the unit in a poorly ventilated area.
- 3. Do not touch the nozzle when the unit is in operation.

**How to use**

**Checklist**

- 1. Pump must be primed before liquid can be ejected.
- 2. Do not press the lever too strongly. Pressing too hard may cause the dispenser to malfunction.
- 3. Place the warning label on the bottle before the dispenser is in operation.

**Maintenance**

**Checklist**

- 1. Clean the nozzle with a soft cloth.
- 2. Do not use alcohol disinfectant spray on the nozzle.
- 3. Do not touch the nozzle when the unit is in operation.

**Maintenance**

**Checklist**

- 1. Investigate the points above if the unit does not dispense.
- 2. Contact your sales representative if you are unsure about requiring maintenance or if other problems should occur.

**Specifications**

Item	Details
Name	Elbow Operated Dispenser
Model	MDS 1000A 1000P 500A 500P
Dimensions (mm)	W594xD218xH420 W594xD218xH420
Weight	Approx. 700g Approx. 400g
Level	RS/BS/SA
Leak cover	PP
Body	AL PP AL PP
Pump	PP/BS
Dispensing volume	SARAYA Liquid Soap, SARAYA Disinfectant
Capacity	1000mL 500mL
Dispense rate	1.5mL per dispense (approx.)

**Installation**

**Checklist**

- 1. Pump must be primed before liquid can be ejected.
- 2. Do not press the lever too strongly. Pressing too hard may cause the dispenser to malfunction.
- 3. Place the warning label on the bottle before the dispenser is in operation.

**Installation**

**Checklist**

- 1. Ensure the unit is mounted on a concrete wall.
- 2. Do not use the unit in a poorly ventilated area.
- 3. Do not touch the nozzle when the unit is in operation.

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**Checklist**

- 1. Pump must be primed before liquid can be ejected.
- 2. Do not press the lever too strongly. Pressing too hard may cause the dispenser to malfunction.
- 3. Place the warning label on the bottle before the dispenser is in operation.

**Maintenance**

**Checklist**

- 1. Clean the nozzle with a soft cloth.
- 2. Do not use alcohol disinfectant spray on the nozzle.
- 3. Do not touch the nozzle when the unit is in operation.

**Maintenance**

**Checklist**

- 1. Investigate the points above if the unit does not dispense.
- 2. Contact your sales representative if you are unsure about requiring maintenance or if other problems should occur.

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